## C-Pro<sup>™</sup> 3D





Made of 100% collagen; DeRoyal's C-Pro 3D wound care dressings are designed to help facilitate moist wound healing and improve patient outcomes. C-Pro 3D collagen provides excellent absorption and 3D scaffolding for cell proliferation and migration, in as little as 3 days.







## **C-PRO™ 3D** collagen wound dressing



## features & benefits:

- Made of 100% medical grade bovine collagen
- Provides excellent absorption and scaffolding for cell proliferation and migration
- 3D scaffold structure facilitating tissue repair
- Soft and highly flexible for easy application and superior conformity
- · Creates a soft conformable gel on contact with wound fluid
- Crosslinking process increases mechanical strength and moderates enzymatic digestion
- High exudate absorption and retention provides moist wound healing environment

# Product # Foam Qty/Bx Qty/Cs HCPC 46-CP32 2x2 10 30 A6021

#### indications:

Partial and full-thickness wounds Tunneled / undermined wounds Surgical wounds Low-to-moderately exuding wounds Diabetic foot ulcers Venous leg ulcers Pressure sores Post debridement for hemostasis

## 

### directions for use:

- 1. If necessary, clean/flush wound with saline or other noirritating solution.
- 2. Choose the appropriate size dressing for the wound. It should not extend beyond the wound margin. If necessary, cut the dressing to size.
- 3. Remove dressing from package.
- 4. Place directly into the wound bed or on the wound surface
- 5. Collagen will dissolve into the wound bed. Note: collagen should not be allowed to dry out.
- 6. Cover and secure the dressing with secondary dressing; such as Covaderm or Polyderm.

#### precautions/contraindications:

Third degree burns Dry Wounds Wounds with necrosis Sensitivity to bovine products

DeRoyal®

Improving Care. Improving Business:

DeRoyal Industries, Inc. | 200 DeBusk Lane, Powell, TN 37849 USA 800.DEROYAL or 865.938.7828 | www.deroyal.com Reprint # 0-2373 | Rev. 01/15