


TroClean™

Trocar Cleaning Device

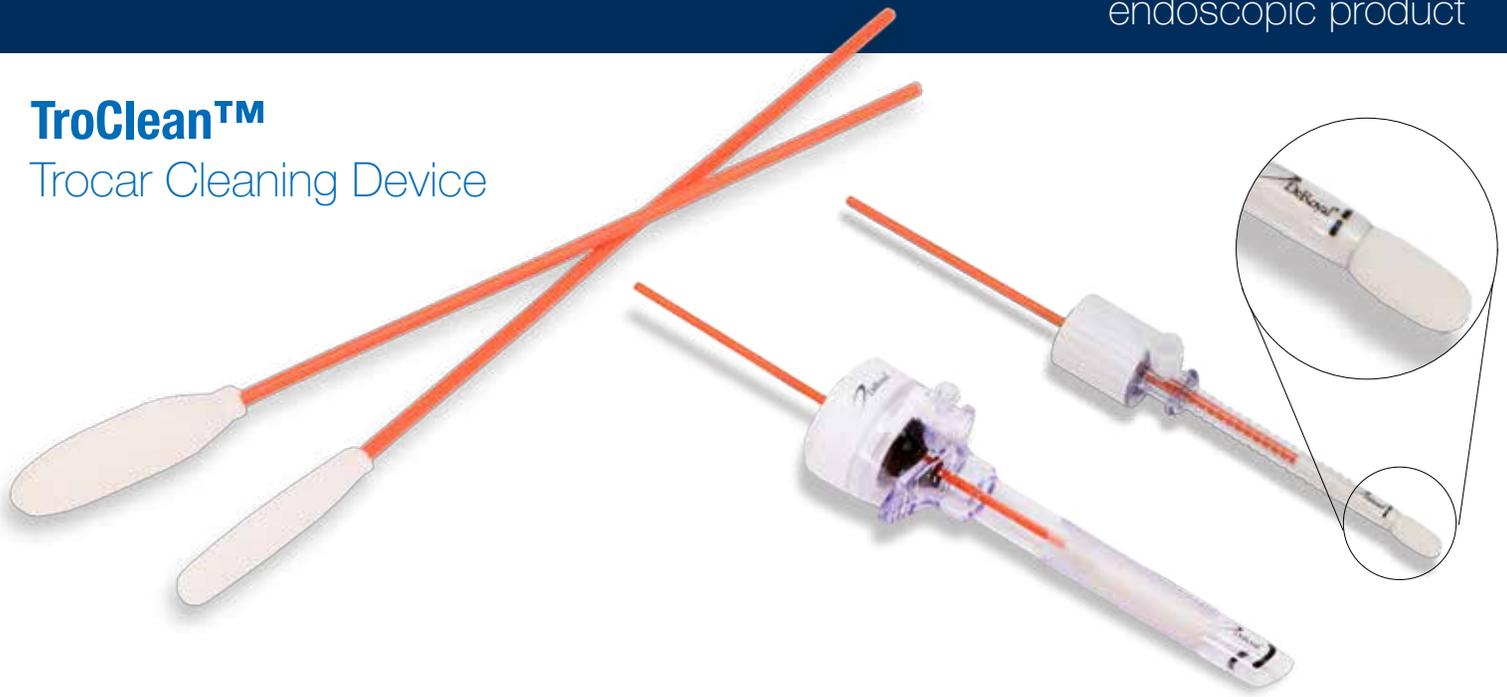

The DeRoyal® TroClean™ Trocar Cleaning Device is a single patient use, sterile application and is used to clean the inside barrel of an endoscopic/laparoscopic trocar before and during the procedure.

- Disposable, ready to use
- Absorbs fluid and removes debris from the inside barrel of the trocar
- Cleans without leaving smudges, biomass, or particulate
- Helps restore vision quickly
- Sterile, single use device

The TroClean™ cleaner offers a small and a large bore cleaning sponge with two different sticks. The small sponge is for 5–7 mm trocars and the large is for trocars 7–12 mm. Use the proper size sponge with the correct trocar size for ease of use. The TroClean™ cleaner is designed to firmly go into the cannula of the trocar while offering a malleable handle for ergonomics and flexibility when cleaning. The TroClean™ cleaner handle is x-ray detectable, but do not deliberately drop device into the patient. If the trocar system is spring loaded, open the valve for ease of insertion and retraction of TroClean™ cleaner. Be aware of any sharp edges when using TroClean™ cleaner and only insert TroClean™ cleaner to the end of the barrel of the cannula. Discard the TroClean™ cleaner post-surgery and follow local, state, and federal guidelines regarding medical waste disposal to avoid any potential cross-infection, environmental contamination, or injury.

TroClean™ Trocar Cleaning Device

28-1002	TroClean™ Trocar Cleaning Device	25 / Case
28-1002K	TroClean™ including DeFogger™ and towel	15 Kits / Case

DeRoyal, the DeRoyal logo, Improving Care, Improving Business, are registered trademarks of DeRoyal Industries, Inc.

TroClean and DeFogger are trademarks of DeRoyal Industries, Inc.

© 2017, DeRoyal. All rights reserved.

DeRoyal Industries, Inc.
200 DeBusk Lane | Powell, TN 37849
1.800.DEROYAL or 865.938.7828

www.deroyal.com | Rev. 01/17 Reprint #0-2408



Improving Care. Improving Business.®