



BACK SUPPORT, SACRAL, CRISS-CROSS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warning and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

PRODUCT DESCRIPTION

The DeRoyal® Lumbo Sacral Back Brace contours for maximum comfort. Malleable stays contour to curve of patient's hip and back with hook and loop closure.

INDICATIONS

Mild low back pain; Lumbar muscle weakness; Lumbar sprains and strains; Mechanical or discogenic thoracolumbar pain

INTENDED USE

The DeRoyal Lumbar Sacral Support is intended to help provide warmth, compression and mild support for lower back pain.

⚠️ WARNINGS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Inspect product for damage and/or missing components before use.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.**
- Prolonged usage can result in muscle atrophy in the user.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace.
- This device is for single patient use only. **DO NOT** reuse on another patient, reprocess, or sterilize.

INSTRUCTION FOR USE

NOTE: Clothing should be worn underneath this brace.

1. Wrap support around body so that the compression straps extend from the middle of the back.
2. Fasten in front with hook and loop.
3. Fasten compression straps down with hook and loop. This allows compression over the lower back area providing added support. **NOTE:** Support should be snug, but comfortable.

CARE INSTRUCTIONS

Hand wash with mild soap and let air dry completely prior to re-application.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#0-037 | REVISED 3/2023

©2023 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.