



Improving Care. Improving Business.®

## WRIST AND THUMB SPLINT

	<b>SINGLE PATIENT USE</b>
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

### INTENDED USE


The DeRoyal® Wrist Splint is intended to provide support, compression and alignment to the wrist.

### INDICATIONS

De Quervain's Syndrome; Scaphoid injuries; Carpometacarpal instabilities; Metacarpal fractures; Post cast healing; Gamekeeper's thumb; Cumulative trauma injuries

### CAUTIONS







- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the wrist/arm (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have

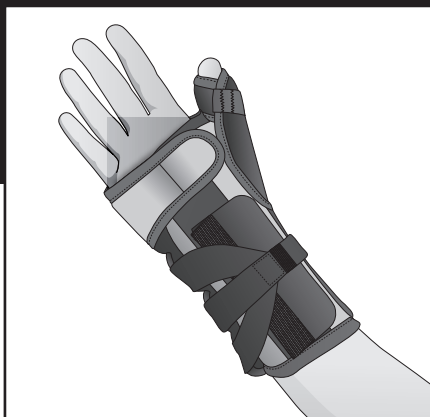

decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.



### INSTRUCTIONS FOR USE

1. Open dynamic, hand and thumb closure to expose the interior portion of the splint.
2. Place hand in splint and slide hand through the splint and thumb through thumbhole.
3. Position splint with palmar side of the splint below palmar crease. Readjust as necessary.
4. Adjust metal wrist and thumb stays for proper positioning and fit.
5. Wrap thumb strap around thumb for snug but comfortable fit.
6. Next, attach the dynamic closure on opposite side of wrist using hook attachment and attach hand closure to desired level of comfort.
7. Once dynamic and hand closure is secured, pull and stretch the dynamic tab to desired level of comfort and secure on opposite side.

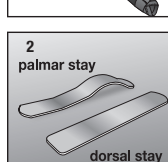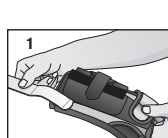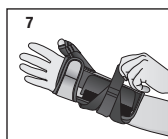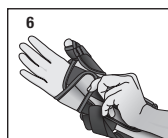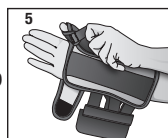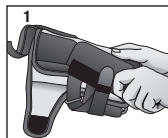

### UNIVERSAL ADJUSTMENT

1. Remove palmar (curved) stay from their pockets and switch them into the opposite pocket. The splint is now set up to wear on the opposite hand. **NOTE:** Stay pockets are held closed by hook dots.
2. Continue with steps 2-7 for fitting.

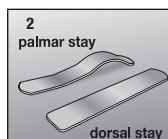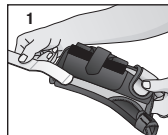



Improving Care. Improving Business.®

### CARE INSTRUCTIONS

Remove the stays before washing. Hand wash in lukewarm water with mild detergent. Air dry completely prior to re-application.

### STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

### WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufactured by:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
[www.deroyal.com](http://www.deroyal.com)

Part#0-1849 Revised 11/2021

©2021 DeRoyal Industries, Inc.  
All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.