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## FIXED POSITION CUFFS

**RX ONLY  
SINGLE PATIENT USE  
NON-STERILE  
NOT MADE WITH NATURAL RUBBER LATEX**

### IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

### INTENDED USE

The DeRoyal Fixed Position Cuffs are intended to be prescribed by a properly licensed practitioner to protect the patient from harming themselves, other patients, family members and/or staff.

### CONTRAINDICATIONS

Never use on a patient if an I.V. or wound site could be compromised by the device.

### WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Secure straps to a portion of the bed or gurney that moves with the patient. Do not secure straps to bed or gurney side rails.
- Keep straps out of patient's reach. Patient should not be able to remove device.
- Make sure bed or gurney side rails are up while product is in use.
- Check the patient regularly to ensure circulation is not compromised.
- Inspect product for proper fit immediately following patient application.
- Discontinue use of device and consult physician or properly licensed practitioner if device becomes damaged in any way, limiting its ability to perform as intended.
- Inspect product for damaged or missing components prior to patient application.

### PRECAUTIONS

This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

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### INSTRUCTIONS FOR USE

- Position the straps lying horizontally across the opposite ends of the bed, approximately where the patient's wrists and ankles will be resting.
- Secure the straps to the bed using quick release buckles. Release the female end of the buckle and wrap its strap around the anchor.
- Pull the female portion through the loop and connect the male and female ends of the buckle.
- Pull the excess strap to tighten.
- Secure the cuffs on the patient.
- Wrap the neoprene cuff around the patient's wrist with the neoprene side first, and then follow with the neoprene side with loop.
- Secure the hook side of the cuff to the loop.
- Apply the cuffs in the same manner for the ankles.

Application videos are available on [www.Derooyal.com](http://www.Derooyal.com).

### SIZING

Style	Cuff Dimensions	Length between Cuffs
Wrist	16" L x 3" W (40 x 7 cm)	22" (56 cm)
Ankle	16" L x 3" W (40 x 7 cm)	15" (38 cm)

### CLEANING AND/OR MAINTENANCE

Prior to washing, remove any detachable rigid stays, fasten any buckles and tie any laces to minimize damage during wash and dry cycles. Close any hook and loop fasteners before washing to help prevent lint build up and pilling. For maximum product life and to help prevent any straps from tangling, wash in a mesh laundry bag. Hand or machine wash with a mild fabric detergent in warm water up to 60°C / 140°F. Bleach as directed on container. Do not iron. Lay flat and open and allow to air dry, or tumble dry on low heat for up to 40 minutes. Inspect the product after washing to ensure it is intact and replace any previously removed stays. Hook and loop fasteners may collect lint after repeated use or laundering, reducing grip strength. As needed, use a stiff bristle brush to remove lint from the "hook" side. Test hook and loop adhesion before use and discard product if hook and loop does not hold securely. During the washing, metal buckles could bend and plastic quick release buckles could crack or break. Always ensure that all buckles are in good working order and are not bent, cracked or broken before each use. If the product is damaged, do not use it and replace it with a new one. Customers could expect up to 5% shrinkage in cotton materials over the life of the product.

### WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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All Rights Reserved.  11/2014