Explanation of Symbols

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Explication des symboles

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Verklaring van de symbolen

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SELF-RELEASING WHEELCHAIR BELT

**RX ONLY
SINGLE PATIENT USE
NON-STERILE
NOT MADE WITH NATURAL RUBBER LATEX**

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal Self-Releasing Wheelchair Belt is intended to be prescribed by a properly licensed practitioner to be used to help keep the patient positioned in wheelchair and prevent forward sliding and getting up unassisted.

CONTRAINDICATIONS

- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient if an I.V. or wound site could be compromised by the device.

WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Always secure straps to a portion of the bed or wheelchair that moves with the patient. Do not secure straps to bed side rails.
- Always keep straps out of patients reach.
- Belts should be worn over patient's clothing.
- Make sure side rails are up while product is in use.
- Check the patient regularly to ensure circulation is not compromised.
- Inspect product for damaged or missing components prior to patient application.
- Inspect product for proper fit immediately following patient application.
- Discontinue use of device and consult physician or properly licensed practitioner if device becomes damaged in any way, limiting its ability to perform as intended.

PRECAUTIONS

This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

- Place the padding of the belt across the patient's waist. **NOTE:** Patient should be sitting up while belt is being applied.
- Feed the straps between the wheelchair armrests, bringing

them behind the wheelchair.

3. Unfasten both buckles. Take the female end of the buckle and wrap its strap around the wheelchair tilt bar. Connect the male and female ends of the buckle. Adjust the slide buckle on the strap to tighten. Repeat this to the other strap. **NOTE:** Secure excess slack in straps.
4. Do not over-tighten. Two fingers should fit between the device and the patient to ensure circulation is not compromised.

Application videos are available on www.deroyal.com.

SIZING

Universal

CLEANING AND/OR MAINTENANCE

Prior to washing, remove any detachable rigid stays, fasten any buckles and tie any laces to minimize damage during wash and dry cycles. Close any hook and loop fasteners before washing to help prevent lint build-up and pilling. For maximum product life and to help prevent any straps from tangling, wash in a mesh laundry bag. Hand or machine-wash with a mild fabric detergent in warm water up to 60°C / 140°F. Bleach as directed on container. Do not iron. Lay flat and open and allow to air dry, or tumble dry on low heat for up to 40 minutes. Inspect the product after washing to ensure it is intact and replace any previously removed stays. Hook and loop fasteners may collect lint after repeated use or laundering, reducing grip strength. As needed, use a stiff bristle brush to remove lint from the "hook" side. Test hook and loop adhesion before use and discard product if hook and loop does not hold securely. During the washing, metal buckles could bend and plastic quick-release buckles could crack or break. Always ensure that all buckles are in good working order and are not bent, cracked or broken before each use. If the product is damaged, do not use it and replace it with a new one. Customers could expect up to 5% shrinkage in cotton materials over the life of the product.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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All Rights Reserved.