



Improving Care. Improving Business.®

QUICK-CHECK MITT WITH BREATHABLE INSPECTION FLAP

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings and precautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Quick-Check Mitts are intended to be prescribed by a properly licensed practitioner to aid in controlling hand and finger or limb movements to prevent patients from disturbing medical treatment.

CONTRAINDICATIONS


- Never use on a patient if an I.V. or wound site could be compromised by the device.
- **DO NOT** use on a patient who is or may become highly aggressive or agitated

WARNINGS

- Improper application or use of any mitts may result in serious injury or death.
- Check to ensure that mitts are properly secured and circulation is not compromised. If applied too loosely, the patient may be able to release or remove themselves from the device
- Check to ensure mitts are not damaged, missing components, or contaminated, and that all hook and loop and/or buckle closures function properly.
- Use hospital approved bed safety guidelines. If securing the device to a bed or chair, always secure straps to the portion of the bed or chair that moves with the patient. **DO NOT** secure straps to bed side rails. Always keep straps out of patient's reach. Check the patient regularly per hospital's policies and procedures to ensure circulation is not compromised.

CAUTIONS

- This product is to be fitted a properly licensed practitioner who is familiar with the purpose for which it is intended. The practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care.
- The practitioner must be trained and demonstrate competency in using this device per the instructions provided and per facility policies,



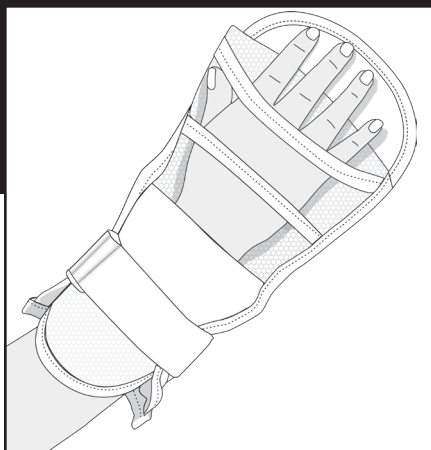


state and federal regulations. The practitioner should perform a complete assessment of the patient to ensure the device is necessary. Use a restraint only when all other alternative methods of treatment and intervention have failed to meet the patient's needs.

- Be sure to follow your facility's policies and guidelines for frequency of patient monitoring and documentation.
- Patients who ambulate with this product are at risk of a fall.

INSTRUCTIONS FOR USE

1. Release the hook strap and open the wrist closure.
2. Place the patient's hand in the mitt with the padded side touching the patient's palm.
3. To secure wrist closure, pull the hook and loop strap until the wrist closure fits snug around the patient's wrist. Secure the hook strap to the loop side of the strap. **DO NOT OVERTIGHTEN**, two fingers should fit between the device and the patient to ensure circulation is not compromised.
4. To open the inspection flap, pull the flap from the underside of the mitt until the mitt opens to expose fingers. To close the inspection flap, pull the flap up and over the fingers to the underside of the mitt.
5. If the mitt is used with an optional bed attachment strap, feed the end of the strap through the loop on the wrist closure of the mitt. Then feed the end of the strap through the seawave buckle connected to the strap and pull so that the buckle is positioned near the cotton loop on the wrist closure.
6. To secure the strap to the bed using quick release ties, wrap the strap(s) around the anchor

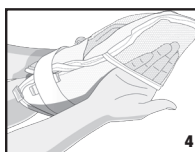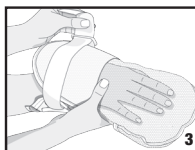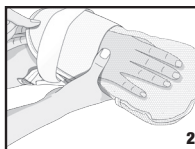



Improving Care. Improving Business.®

once. Take the loose end of the strap(s) and create a loop. Cross the loop over the other strap(s) that is wrapped around the anchor and feed it through opening as if tying a shoelace. Hold that loop and create another loop with the loose strap(s) and feed it through the first loop. Pull the final loop and the strap(s) attached to the cuff in opposite directions to form a quick release tie. Secure excess strap out of the patient's reach

CLEANING AND/OR MAINTENANCE (IF APPLICABLE)

Please refer to hospital approved guidelines for cleaning and disinfecting of the device.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufacturer:

DeRoyal Industries, Inc.

200 DeBusk Lane

Powell, TN 37849 USA

888.938.7828 or (001) 865.938.7828

www.deroyal.com

PART#0-1930 REVISED 2/2020

©2020 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.