



Improving Care. Improving Business.®

PRUVENTOR™ TURNPRO™ KIT TURNING AND POSITIONING SYSTEM

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® PRUventor™ TurnPRO™ System is intended to assist and maintain proper positioning of the patient while helping prevent sacral pressure ulcers.

The sheet is intended to provide assistance to the caregiver when repositioning the patient.

The wedges are intended to off-load the sacrum.

The absorbent pad is intended to help control body heat through breathable material and moisture-wicking technology.

CONTRAINDICATION


Patients who have an unstable spinal injury should not use this system, unless a decision to use has been made by the healthcare professional.

WARNING

- Prior to use, inspect the kit components for damaged, missing, or contaminated product.
- Consult physician or properly licensed practitioner immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this device.
- Do not attempt to carry patient while using this device.
- Do not hold onto the absorbent pad as a method to position the patient.
- Prior to positioning a patient, ensure movement will not disrupt treatment.
- Make sure patient's head does not drag during positioning.

CAUTIONS

- The components within this system have only been tested in conjunction with each other. Use of outside components could alter proper performance.
- Make sure the pad and sheet lay flat and smooth beneath the patient.
- Make sure handles are always draped away from the patient.
- If the patient can assist during the application process, communicate to assist as necessary prior to application.
- Be sure to follow your hospital's policies and procedures for frequency of patient positioning and monitoring.

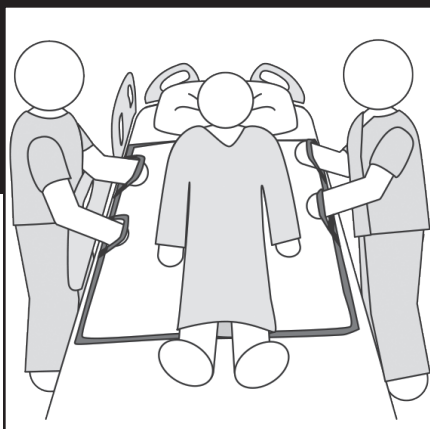

INSTRUCTIONS FOR USE


Follow your facility's policies and procedures for safe patient handling.

PREPARE THE BED

1. Make sure bed brakes are in the locked position.
2. Make sure the bed is as flat as possible (if it does not interfere with the patient's condition).
3. Lower or raise the bed to ensure the top of the mattress is at your waist level or elbows.

APPLYING THE SHEET W/ ABSORBENT PAD

1. Make sure all side rails are in the up position.
2. Lower the side rail(s) on the side the clinician is standing to apply the sheet.
3. Center the folded sheet and pad alongside the patient on the bed.
4. Unfold the sheet and pad along the side of the patient lengthwise, position top of the sheet directly above patient shoulders.
5. Roll patient away from the clinician onto their side and keep them side-lying (may require up to 4 individuals to roll patient).
6. Unfold the sheet and pad to expose center. Open the side closest to clinician and drape the handles of the sheet over the edge of the bed.
7. Push and tuck the folded side of the sheet beneath the patient, making sure the top of the sheet sits directly above the patient's shoulders.
8. Raise the side rail(s), and gently roll the patient onto their back.
9. Lower the side rail(s) on the other side.
10. Gently roll the patient onto the exposed sheet and pad and keep them side-lying (may require up to 4 individuals to roll the patient).
11. Unfold the rest of the sheet and pad. Drape handles of the sheet over the edge of the bed.
12. Raise the side rail(s), and gently roll the patient onto their back.
13. As needed, use the handles lining the border of the sheet, to position the patient.


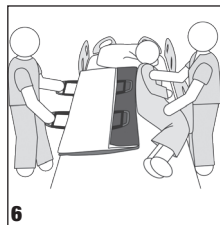




Improving Care. Improving Business.®

APPLYING THE WEDGES

1. Make sure all the side rails are in the up position.
2. Take the two long handles and drape them over the patient to the opposite side of the bed.
3. From the opposite side of the bed, hold onto both handles and pull, to roll the patient onto their side. If needed, utilize short handles to hold this position.
4. Make sure the patient is completely side-lying and locate the sacrum. **NOTE:** Sacral locator sticker can be applied to assist in proper wedge application.
5. Fully insert the first wedge above the sacrum to support the upper torso.
6. Fully insert the second wedge below the sacrum to support the hips, buttocks, and thighs.
7. Gently roll patient onto their back, making sure sacrum is completely off-loading and re-adjust as needed.
8. Smooth out the sheet and pad, and drape handles over the edge of the bed.

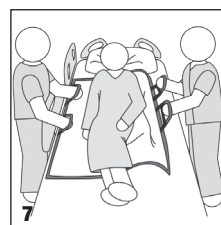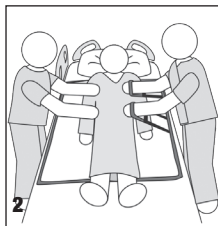

CLEANING INSTRUCTIONS

ABSORBENT PAD - If the pad is soiled, dispose and replace promptly, in accordance with hospital protocols and guidelines.

SHEET - If the sheet is soiled wipe down with a damp cloth or hospital-approved disinfectant. Dry completely prior to use. If the product is damaged or cleaning is insufficient, replace with a new one.

WEDGES - If the covered portion of the wedge is soiled wipe down with a damp cloth or hospital-approved disinfectant. Dry completely prior to use. If the product is damaged or cleaning is insufficient, replace with a new one.

Replacement sheets, wedges, and pads are available.

SIZING

Universal Sheet (35" x 55")
X-Large Sheet (45" x 55")

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufacturer:
DeRoyal Industries, Inc.

200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or 865.938.7828
www.deroyal.com

Part#0-2010 Revised 1/19
©2019 DeRoyal Industries, Inc.

All Rights Reserved. PRUventor, TurnPro, DeRoyal, the DeRoyal logo and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.