

DeRoyal® PRUventor™

APPLICATION GUIDE

The PRUventor™ has a soft interior filling that when combined with the unique fabric, creates a fluid-like sensation. Therefore, pressure around the bony prominences of the ankles and metatarsals is reduced.



APPLICATION INSTRUCTIONS



Steps for Applications:

1. Open all strap closures and secure the hook tabs to the loop on either side of the boot. The DeRoyal sew-in label should be on the outside of the boot.
2. Gently place the patient's foot into the boot, making sure the heel is placed over the heel opening.
3. Take one strap and laterally attach to the same side of the boot. Secure hook and loop on the same side of the boot.
4. Take the other strap and laterally attach to the same side of the boot. Secure hook and loop on the same side of the boot.
5. Take the lower leg straps and apply the hook tab(s) to the loop on either side of the boot. Caregiver can choose to use one or both straps to assist in holding boot on patient's foot by suggested strapping configuration shown above.
6. Make sure two fingers fit between the device and the patient to ensure circulation is not compromised. Pull flaps on either side of the foot, toward the patient to ensure the product fits securely and that the heel is suspended over the heel opening.
7. Make sure the heel is suspended. Check the bottom of the boot to ensure that nothing is touching the heel.
8. If wedge is included: To apply the anti-rotation wedge, place the hook side of the anti-rotation wedge on the outside, making sure that the bottom of the wedge sits flat on the surface of the bed.

NOTE: Perform periodic skin assessment as specified by hospital policies and procedures.

Application videos are available on www.deroyal.com.



Improving Care. Improving Business.®