



PATIENT EYE SHIELD

	DO NOT RESTERILIZE
	DO NOT REUSE
STERILE/EO	STERILIZED WITH ETHYLENE OXIDE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.



IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

DESCRIPTION

The DeRoyal® Patient Eye Shields are lenses backed with an adhesive that are to be put over the eyes of a patient undergoing a medical procedure. The Patient Eye Shield offers maximum protection by covering and protecting patient's eyes from accidental contact.

INTENDED USE

The DeRoyal® Patient Eye Shields are intended to protect the patient's eyes from inadvertent contact during medical procedures.

CONTRAINDICATION

DO NOT use on neonatal patients.

WARNINGS

- DO NOT** use if the individual package is opened or if the package or eye shield is damaged.
- This is a single use device. It is not intended for disinfection and/or subsequent re-use. Re-use could result in microbial contamination and potential patient injury.
- This device has not been evaluated for reprocessing or re-sterilization. Reprocessing and/or re-sterilization may damage the device, rendering it unusable and/or may lead to device failure.
- If there is swelling, redness or signs of an allergic reaction immediately discontinue use of the product.
- DO NOT** use on damaged, irritated, fragile, or sensitized skin.
- DO NOT** allow adhesive to contact eyes.

- DO NOT** use on pediatric or other patients if eyes are not aligned properly under the clear lens.
- DO NOT** use in the prone position.
- DO NOT** place any objects on top of eye shield.

DIRECTIONS FOR USE

- Open the package using aseptic technique.
- Peel both halves of the paper liner from the back of the eye shield to expose the adhesive.
- Position eye shield over the eyes. The patient's eyes should be aligned under the clear lens so they remain visible.
- Once in position gently press to ensure adhesive adheres to the intact skin.
- To remove, gently and carefully lift and peel eye shield away from the skin. If needed, isopropyl alcohol can be used to assist in releasing the adhesive from the patient's skin. **NOTE:** Avoid contacting the patient's eyes with isopropyl alcohol if used.
- Dispose of device according to hospital protocol.



STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

DISPOSAL

Dispose of any used device according to local, state, and federal laws and regulations. For safe disposal of devices, follow your facility's protocol.



In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#0-405 | REVISED 6/2024

©2024 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.