




SELF-RELEASING ROLL BELTS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

 Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product. Use only on the patient that the device was provided to by a healthcare professional. The device is only to be used as originally intended.

INTENDED USE

The DeRoyal® Self-Releasing Roll Belt is intended to be prescribed by a properly licensed practitioner to allow patients to roll from side-to-side and sit up in bed while preventing them from falling or getting up unassisted.

CONTRAINDICATIONS

- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient whose I.V. or wound site could be compromised by the device.

WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Always secure straps to a portion of the bed that moves with the patient. Do not secure straps to bed or gurney side rails.
- Always keep straps out of patient's reach.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure breathing and circulation are not compromised.

CAUTIONS

- A physician or properly licensed practitioner (a "Prescriber") who is familiar with the use and purpose of this product must fit it to the user. The Prescriber has a duty to provide wearing instructions and risks related to the use of this product to other



healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the Prescriber. Use only as directed.

- Inspect product for damaged or missing components before use.
- Discontinue use and consult your Prescriber if the product or its components break or become damaged, or cleaning is no longer sufficient to properly sanitize product.
- Belt should be worn over patient's clothing.
- Consult Prescriber immediately if patient experiences sensation changes, unusual reactions, swelling or increased pain while using this product. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

INSTRUCTIONS FOR USE

1. Position the patient in an upright position on the bed. Place the belt on the center of the bed at the patient's waist line.
2. Position the patient on his/her back and wrap the belt around the waist.
3. Secure the belt with hook and loop closure. **DO NOT OVERTIGHTEN.** Two fingers should fit between the device and the patient to ensure that circulation is not compromised.
4. Secure the quick-release buckle and tighten the strap on the front of the belt.
5. Secure the straps to the bed with quick-release ties. Wrap the strap(s) around the anchor once. Create a loop using the loose end of the strap(s). Cross the loop over the other strap(s) that is wrapped around the anchor and feed it through opening as if tying a shoelace. Hold the existing loop and



create another loop with the loose strap(s) and feed it through the first loop. Pull the final loop and the strap(s) attached to the cuff in opposite directions to form a quick release tie.

- Secure excess strap out of the patient's reach.

SIZING

Universal 49"L x 6"W (125 cm x 15 cm)

Bariatric 60"L x 6"W (150 cm x 15 cm)

CLEANING AND/OR MAINTENANCE

Prior to washing, fasten any buckles and tie any laces to minimize damage during wash and dry cycles. Close any hook and loop fasteners before washing to help prevent lint build up and pilling. For maximum product life and to help prevent any straps from tangling, wash in a mesh laundry bag. Hand or machine-wash with a mild fabric detergent in warm water up to 60°C / 140°F. Bleach as directed on container. Do not iron. Lay flat and open and allow to air dry, or tumble dry on low heat for up to 40 minutes. Inspect the product after washing to ensure it is intact. Hook and loop fasteners may collect lint after repeated use or laundering, reducing grip strength. As needed, use a stiff bristle brush to remove lint from the "hook" side. Test hook and loop adhesion before use and discard product if hook and loop does not hold securely. During the washing, metal buckles could bend and plastic quick-release buckles could crack or break. Always ensure that all buckles are in good working order and are not bent, cracked or broken before each use. If the product is damaged, do not use, and replace with a new product. Customers may experience up to 5% shrinkage in cotton materials over the life of the product.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#0-745 | REVISED 6/2025

©2025 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.