## PRO-GLIDE™ DYNAMIC ELBOW ROM ORTHOSIS

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Manufactured by:  
DeRoyal Industries, Inc.  
200 DeBak Lane  
Powell, TN 37849 USA  
888.938.7228 or (901) 865.938.7228  
www.derooyal.com

EMERGO EUROPE  
Prinsengracht 20  
2514 AP The Hague  
The Netherlands

**Part #1-4125G**  
Revised 2/2021  
©2021 DeRoyal Industries, Inc.  
All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care, Improving Business, Wire-foam, Pro-Glide, and Pro-Lock are trademarks or registered trademarks of DeRoyal Industries, Inc.

## PRO-GLIDE™ DYNAMIC ELBOW ROM ORTHOSIS

	<b>SINGLE PATIENT USE</b>
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all instructions, warnings and cautions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

### INTENDED USE

The Pro-Glide™ Dynamic Elbow ROM Orthosis was designed for the treatment of joint stiffness and established soft tissue contractures which may have resulted from trauma or immobilization. Protocols for use vary depending on the diagnosis and should be determined after consulting with the patient's physician. This orthosis combines flexion and extension modes in a single unit. Each orthosis is delivered in an extension mode. Conversion to flexion is described in detail in step 5.

### CONTRAINDICATIONS

Not to be used if any of the following conditions exist: Severe spasticity; severe osteoporosis; thrombophlebitis. Diabetes requires frequent skin checks.

### CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a wearing period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the elbow (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation

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changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.

- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

### FORCE ADJUSTMENTS

ADJUSTMENT	FORCE INDEX IN WINDOW					
<b>ARM LENGTH</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>
Minimum	3.0	4.5	6.5	9.0	11.0	13.0
Medium	2.7	4.0	5.9	8.1	9.9	11.7
Maximum	2.5	3.7	5.4	7.4	9.1	10.7

**NOTE:** Force decreases as adjustment arm length increases. Force readings are in pounds and are accurate to within +10%. Minimum adjustment arm length = 7.0" of leverage. Medium adjustment arm length = 7.75" of leverage. Maximum adjustment arm length = 8.5" of leverage. Leverage is measured from axis (Pro-lock™ Holder) to end of adjustment arm (step 2).

### INSTRUCTIONS FOR USE

- ADJUST/FIT CENTER CUFF:** Lay splint open with all straps unfastened. Loosen screws and adjust width of center cuff by adjusting brackets to fit forearm. Cuff must be centered; if not, loosen screws and try moving again. Tighten screws securely.
- ADJUST/FIT DISTAL CUFF:** Insert arm into splint with elbow crease aligned with pivot point of hinge. **NOTE:** The distal cuff may be adjusted left or right if necessary. Loosen screws and set length of adjustment arm, positioning the distal cuff 1" proximal to the head of the ulna. Tighten screws. **NOTE:** Length of adjustment arm will determine the amount of force, see Force Adjustments chart. Wrap forearm with cuff and secure with distal straps. **NOTE:** The strap end with the burgandy stripe is used for quick donning and doffing. Clinch distal straps from opposite end and trim excess strap material with scissors. **DO NOT** cut end with the burgandy stripe.
- ADJUST/FIT – PROXIMAL CUFF:** With arm in splint, position the proximal cuff just distal to the armpit, but not so close as to make it uncomfortable. If further adjustment is necessary, remove foam liner, remove the two screws, and relocate the cuff to the desired adjustment holes. Refasten cuff and insert liner.
- ADJUST TENSION:** The force is increased or decreased by turning either end of the force adjustment screw. Use the side that allows easiest access with the adjustment tool.
- EXTENSION/FLEXION CONVERSION:** Conversion of the orthosis from extension is accomplished by turning the force adjustment screw and reversing the tension on the cable. Continue turning until no slack is present in the cable and the desired tension has been reached. **NOTE:** When the cable is wrapped around the bottom portion of the pulley the orthosis is in extension, when wrapped around the top portion of the pulley the orthosis is in flexion.
- PRO-LOCK™ EZ APPLICATION SYSTEM:** The 3 position Pro-lock™ EZ application system enables the patient to don and doff the orthosis without reducing the tension setting. Remove the Pro-lock™ pin from its storage holder and place in the locking hole. Gradually move arm against the applied tension force while

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pushing downward on the Pro-lock™ pin. The pin will slide into a receiving hole when it is aligned making a "click" sound. The Pro-lock™ pin locks the hinge at 0°, 45°, or 90°. To remove the Pro-lock™ pin, move against the applied tension force to take pressure off the pin. Slide and rotate the pin at the same time out of the receiving hole and insert back into its holder. To achieve more flexion, loosen top screw on forearm bracket, push forearm bracket toward the bicep cuff, tighten screw.

### SPECIFICATIONS

**WEIGHT:** 26 oz./ .74 kg  
**RANGE:**  
• **EXTENSION:** 25° hyper-extended to 130° flexed.  
• **FLEXION:** 130° flexed to 25° hyper-extended.  
**MAXIMUM FORCE:** 13.75 lbs./ 6.23 kg

### CARE INSTRUCTIONS

Clean foam cuffs and exterior of hinge with non-abrasive cleaner. Straps and liners may be machine washed. Air dry splint and straps prior to wearing.

### STORAGE AND TRANSPORT CONDITIONS

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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