## PUCCI® INFLATABLE ELBOW ORTHOSIS

## ORTESIS DE CODO INFLABLE PUCCI®

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## PUCCI® INFLATABLE ELBOW ORTHOSIS

|  | SINGLE PATIENT USE   |
|--|--|
|  | NON-STERILE  |
|  | NOT MADE WITH NATURAL RUBBER LATEX   |
|  | FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER. |

### IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

### INTENDED USE

The DeRoyal® Pucci® Inflation Elbow Orthosis is intended to provide continuous low stretch therapy to the contracted elbow.

### CONTRAINDICATIONS

Severe or fixed contractures; excessive edema or swelling; excessive pain.

### CAUTIONS

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the elbow (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

### INSTRUCTIONS FOR USE

- Measure the circumference of the patient's bicep to determine appropriate size. If the patient's bicep measurement is greater than 8½" (21.6 cm) use a large. If less, use a small.
- Open all hook and loop closures.
- Insert the clear end of the inflator bulb into the air valves on the air bladder, and inflate to the degree needed for treatment. **NOTE:** Air bladder must be inflated before applying orthosis to patient. For contractures equal to or greater than 90°, pump 1-2 times. For contractures between 30° and 90°, pump 3 - 4 times. For contractures equal to or less than 30°, pump 4-6 times.
- Extend the contracted arm as far as comfort will allow. Position the

orthosis on top of the arm, with the fleece side against the arm. Center on the elbow crease with the inflation valve at the bottom, toward the wrist. **NOTE:** Check labeling for locating proper position. **NOTE:** When applying product to a contracted elbow, it may be easier to first fold the orthosis and slide it into fitting position. This approach works best when there is limited space between the biceps and forearm due to the elbow in a flexed position.

- Fasten the wide straps above and below the elbow. **NOTE:** Do not inflate orthosis after the straps are fastened. **NOTE:** Be sure that the orthosis is not too tight by checking that a finger can be inserted between the arm and the orthosis.
- Check radial pulse. If pulse has changed rate/strength, attach the black end of the inflation bulb to the air valve on the air bladder, and release air pressure until pulse returns to the previous rate/strength.
- To attach the elbow pad, position the pad with the fleece side toward the elbow. Insert straps through “D” rings on the orthosis, and fasten the hook and loop closures.
- If using the double upright external hinge/orthosis, attach to the posterior aspect of the arm. **NOTE:** Fit larger cuff on bicep and smaller cuff on forearm. **NOTE:** Add fleece cover to external orthosis on side of elbow next to body (medial side). Fasten the straps over the anterior aspect of the arm.
- Remove the orthosis after 15 minutes of wearing and assess for redness and pressure points. If redness is present, immediately release some of the air pressure from the air bladder and replace the orthosis. Assess again in 15 minutes, followed by hourly checks or follow your facilities' protocol.
- Once air pressure is established, remove and replace the orthosis, without inflating or deflating the bladder.

### CARE INSTRUCTIONS

Clean air bladder and frame with a mild detergent and water. Cover washing instructions: Remove the air bladder and plastic upright stays. Fasten all hook and loop closures. Place in mesh laundry bag. Machine wash on gentle cycle with mild detergent and warm water. **DO NOT** use bleach or fabric softener. Rinse in cold water and air dry prior to wearing. Covers can be washed but not sterilized.

### STORAGE AND TRANSPORT CONDITIONS

|  |                         |
|--|-------------------------|
|  | KEEP DRY                |
|  | KEEP AWAY FROM SUNLIGHT |

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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### CONDITIONS OF STOCKAGE ET DE TRANSPORT

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All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care. Improving Business, and Pucci are trademarks or registered trademarks of DeRoyal Industries, Inc.