HAND SPLINT

FÉRULA DE MANO

ATTELLE DE MAIN

HANDSCHIENE

STECCA PER MANO

TALA PARA MÃO

HANDSPALK

HANDSKENA

EL ATELI

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Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (601) 865.938.7828
www.deroyal.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Part#1-711G
Revised 2/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.

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HAND SPLINT

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The DeRoyal® Hand Splint is designed to prevent hyperextension of the ring and small fingers after an ulnar nerve injury. The design provides a light metacarpophalangeal block to the digits and allows full dynamic function of the hand.

CAUTIONS

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the hand (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

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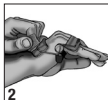
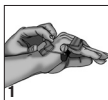
SIZING

Splint size is determined by the width of the hand at the metacarpophalangeal joints.

SIZE	WIDTH	
B	2¼ - 2¾"	5.7 - 7 cm
C	2⅞ - 3¼"	7 - 8.25 cm
D	3⅜ - 3¾"	8.25 - 9.5 cm

INSTRUCTIONS FOR USE

- Slide forth and fifth digits under the dynamic metacarpophalan-geal block. Determine if the wire dynamic metacarpophalan-geal (MP) block is positioned at the proximal phalanges to prevent metacarpophalangeal hyperextension. Adjust the distal wire and bend to the desired position.
- To secure the splint, fasten the strap to the hook material on the dorsum of the splint. Trim excess strapping.

CARE INSTRUCTIONS

Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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