



Improving Care. Improving Business.®

H2No™ SUCTION & ANTI-FATIGUE MAT

	DO NOT REUSE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of product.

INTENDED USE

The DeRoyal® H2No™ Suction and Fatigue Mat is intended to aid in the management and disposal of waste fluid during a variety of medical procedures.

CONTRAINDICATIONS

The DeRoyal® H2No™ Suction and Fatigue Mat is not intended to be used for reintroducing bodily fluid back into the body. This system is not recommended for pleural drainage.

⚠️ WARNINGS



- Inspect the product for visible cracks or broken components.
- **DO NOT** reuse or attempt to sterilize H2No™ Suction and Fatigue Mat.

CAUTIONS

- This device is for single patient use only. It is not intended for disinfection and subsequent re-use, which may result in device failure or create the risk of contamination.
- This device has not been evaluated for reprocessing or re-sterilization. Reprocessing and/or re-sterilization may damage the device, rendering it unusable and/or may lead to device failure, which could result in patient illness, injury or death.

DIRECTIONS FOR USE

1. Remove H2No™ Suction and Fatigue Mat from poly bag and place on a clean and dry floor.
2. Unfold the absorbent mat for control of fluid that is not captured by the H2No™ Suction and Fatigue Mat.
3. Orient the mat where the channels run towards the operating room table and the tubing is closest, running parallel, to the table (Image 1).

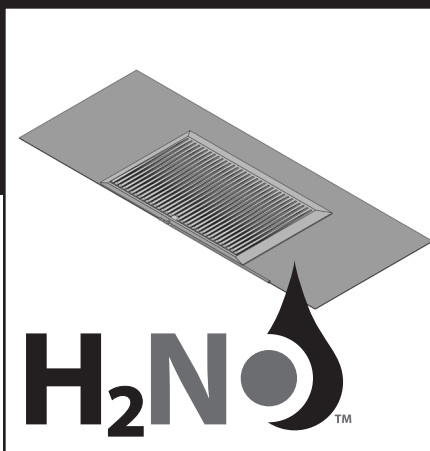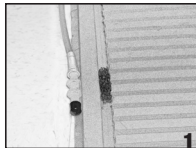

4. Use non-sterile suction tubing to create suction from the mat to the waste fluid management system, canisters or capital equipment.
5. Make sure the mat is positioned to effectively catch the flow of the fluid off of the drapes. The ideal method would be to take a hemostat, or clamp, and position to make a channel for fluid to run smoothly on the mat (Image 2).

STORAGE AND TRANSPORT CONDITIONS

Do not store product at extreme temperatures or in a moist/damp environment.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.



Improving Care. Improving Business.®

Distributed By:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#1007909 Revised 5/2022

©2022 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care. Improving Business., the H2No logo and H2No are trademarks or registered trademarks of DeRoyal Industries, Inc.