## DEFOGGER™ ANTI-FOG KIT

	MEDICAL DEVICE
	SINGLE PATIENT USE
	STERILIZED WITH IRRADIATION
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN OR PROPE LICENSED PRACTITIONER.

### GENERAL

The following information should be read prior to use of this device. The DeFogger Anti Fog Kit is a sterile, single use, disposable anti fog solution and sponge designed for use on scopes and camera lens during laparoscopic, gastroscopic and arthroscopic surgeries.

### INTENDED USE

The DeFogger Anti Fog Kit is used during operative laparoscopy, gastroscopy or arthroscopy, to help prevent the scope or camera lens from fogging.

### CAUTIONS/WARNINGS

The DeFogger Anti Fog Kit is not intended to be used in or around the eyes. If DeFogger comes in contact with the eyes, immediately flush with water and contact a physician.

### DIRECTIONS FOR USE

#### LAPAROSCOPY AND GASTROSCOPY

1. Remove paper backing from sponge and place sponge onto sterile drape.
2. Twist off tube tip and apply five or six drops of DeFogger to sponge.
3. Touch end of scope to sponge, dry excess solution with a sterile sponge.
4. Apply one drop of DeFogger to eyepiece of scope, wipe dry with a sterile sponge.
5. Repeat as necessary.

#### FOR ARTHROSCOPY

1. Apply DeFogger to the camera lens and scope eyepiece.
2. Dry the excess solution with a sterile sponge.

#### STORAGE AND TRANSPORT CONDITIONS

Do not store product at extreme temperatures or in a

moist/damp environment.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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