



Improving Care. Improving Business.®

## SAFETY SCALPEL WITH RETRACTABLE BLADE

	<b>DO NOT REUSE</b>
	<b>STERILIZED WITH ETHYLENE OXIDE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all instructions, cautions and warnings before use. Correct application is essential for proper functioning of the product.

### INTENDED USE

The DeRoyal® Safety Scalpel with Retractable Blade is intended to be used as a cutting device during medical procedures. This device is intended for single use only.

### INDICATIONS


The DeRoyal® Safety Scalpel meets engineering design requirements put forth in the Needle-stick Safety and Prevention Act (HR 5178, November 7, 2000) to protect against injuries and infections from exposed scalpel blades.

### WARNINGS

- If this product is being used along with other manufacturer's components, carefully read their instructions prior to use.
- Follow proper sharps disposal procedure as dictated by your hospital protocol.

### INSTRUCTIONS FOR USE

1. Open package containing DeRoyal® Safety Scalpel using aseptic technique.
2. Grasp the scalpel and carefully extend the blade by moving the blue slide toward the tip of the scalpel, using the thumb of the hand holding the scalpel.
3. Extend the slider until you reach the positive stop, the slider will fit in the notch when it is completely extended.
4. To retract the blade, grasp the scalpel carefully and move the slide toward the back of the scalpel, using the thumb of the hand holding the scalpel.
5. You should feel clicks as the blade is retracted, and a positive stop once the blade is completely retracted.
6. To retract the blade permanently, move the blue slider past the notch at the back of the scalpel.
7. Dispose of the DeRoyal® Safety Scalpel in a puncture-resistant container approved for sharps disposal, or in a manner consistent with your hospital protocol.

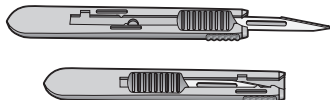

### STORAGE AND TRANSPORT CONDITIONS

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®

Distributed By:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
[www.deroyal.com](http://www.deroyal.com)

Part#4-10432 Revised 8/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.