SPLASH STOP®

MANUFACTURED BY:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

EC REP **MedEnvoy Global B.V.**
Prinses Margrietplantsoen 33 – Suite 123
2595 AM the Hague
The Netherlands

UK Responsible Person
MedEnvoy UK Limited
85, Great Portland Street, First Floor
London, W1W 7LT
United Kingdom

PART#4-18671G | REVISED 10/23

©2023 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Splash Stop are trademarks or registered trademarks of DeRoyal Industries, Inc.

SPLASH STOP®

	SINGLE PATIENT USE
	NON-STERILE
	MEDICAL DEVICE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT
Read carefully and completely before using.

INTENDED USE

The DeRoyal Splash Stop® is intended to be used to collect and contain patient waste.

INDICATIONS

Anywhere fluid waste is collected using a syringe that must be emptied and then contained for disposal.

WARNINGS

- Close and seal Splash Stop prior to moving or transporting to prevent spillage.

PRECAUTIONS

- Carefully read these instructions before using this product.
- If this product is being used along with other manufacturer's components, also read their instructions for use.
- DO NOT** reach into Splash Stop.
- DO NOT** over-fill Splash Stop.
- Splash Stop is **NOT** a sharps container. Do **NOT** place sharps into Splash Stop.
- Maintain Splash Stop in an upright position on a secure and stable surface when in use.

DIRECTIONS FOR USE

- Open the package containing Splash Stop using sterile technique.
- Set Splash Stop on a stable surface, such as a back table.
- Insert the male luer of a syringe containing waste fluid through the top hole of Splash Stop and press

SPLASH STOP®

	SINGLE PATIENT USE
	NON-STERILE
	MEDICAL DEVICE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

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- the syringe nose firmly against lid funnel.
- Slowly empty the syringe contents into the absorbent material located at the bottom of Splash Stop.
 - When the procedure is complete or the contents fill the Splash Stop, remove the adhesive cover from the red biohazard label and secure the label completely over the opening of Splash Stop.
 - Once the label adheres, it should **NOT** be reopened. Reopening can compromise the sealing ability of the biohazard label.
 - Dispose of Splash Stop in a manner that is consistent with your facility's protocol.

STORAGE CONDITIONS

Do not store product at extreme temperatures or in a moist/damp environment.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal products are warranted for one hundred and twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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