Explanation of Symbols

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Explication des symboles

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Verklaring van de symbolen

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DIRECTIONS FOR USE

- Prepare the skin for adhesive pad by removing body oils and hair.
- Remove the Skin Temperature Sensor from its package.
- Peel off release liner and apply sensor with probe cover to patient's skin.
- Connect the temperature probe to the appropriate reusable cable from DeRoyal (FNICU series only.) Connect the cable to the infant radiant warmer.
- Follow the directions for use of the infant radiant warmer.
- When temperature monitoring is completed, carefully remove adhesive cover from patient's skin.
- Disconnect the probe (at the connector) and discard the probe. To disconnect, grasp both connectors firmly and pull. Do not pull on cable or wire.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

CAUTIONS

- Inspect device prior to use for damage.
- Discontinue use if an allergic reaction is observed from use of the adhesive pad cover.
- Consider a less aggressive adhesive pad cover if skin irritation results upon removal of the cover.
- Electric shock hazard. Do not use if wire insulation is damaged or bare wires are exposed.
- If in use during an electrosurgical procedure, take care to ensure that the cable and connector do not get wet and proper techniques are used during electrosurgical procedures to reduce radio frequency interference current and potential burns to the patient.
- If in use during an electrosurgical procedure, an adequate electrosurgical dispersive ground electrode close to the active surgical site should be properly connected.
- If in use during an electrosurgical procedure, do not intertwine the cables, especially the monitor cables with the electrosurgical unit's cables.
- If in use during an electrosurgical procedure, the operation of the infant radiant warmer may be temporarily affected during electrosurgical activations. Unusual temperature readings should be checked.
- The adhesive cover on this probe should never be applied to a skin surface which has been damaged by abrasion, laceration, incision, or burning.

WARNINGS

- Inaccurate temperature readings may result if:
 - Device is damaged
 - Directions for Use are not followed
 - Incorrect placement or application of device
- Device is single patient use only. Reuse of device is prohibited.
- Use of product outside of the intended use is prohibited.
- Failure of adhesive pad during use may result in incorrect temperature readings. Do not continue use until the adhesive pad is replaced.
- Discontinue use and replace device immediately if temperature readings cease during use.
- While adverse reactions are rare, the following have been associated with use of the skin temperature sensor during use or immediately after use: electrical burns and skin irritation or inflammation.

CAUTIONS

- Inspect device prior to use for damage.
- Discontinue use if an allergic reaction is observed from use of the adhesive pad cover.
- Consider a less aggressive adhesive pad cover if skin irritation results upon removal of the cover.
- Electric shock hazard. Do not use if wire insulation is damaged or bare wires are exposed.
- If in use during an electrosurgical procedure, take care to ensure that the cable and connector do not get wet and proper techniques are used during electrosurgical procedures to reduce radio frequency interference current and potential burns to the patient.
- If in use during an electrosurgical procedure, an adequate electrosurgical dispersive ground electrode close to the active surgical site should be properly connected.
- If in use during an electrosurgical procedure, do not intertwine the cables, especially the monitor cables with the electrosurgical unit's cables.
- If in use during an electrosurgical procedure, the operation of the infant radiant warmer may be temporarily affected during electrosurgical activations. Unusual temperature readings should be checked.
- The adhesive cover on this probe should never be applied to a skin surface which has been damaged by abrasion, laceration, incision, or burning.

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NEONATAL TEMPERATURE PROBE

FOR SINGLE USE ONLY DO NOT REUSE NON-STERILE DEVICE

IMPORTANT

- The following information should be read prior to use of this device.
- Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner)

INTENDED USE

The DeRoyal Neonatal Skin Surface Temperature Probe is a non-sterile device intended for connection to an infant radiant warmer to monitor the neonatal patient's skin surface temperature.

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All Rights Reserved.