EN: ELECTROSURGICAL PENCIL AND ELECTRODES

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MANUFACTURED BY: DeRoyal Industries, Inc. 2006 DeRoyal Lane Powell, TN 37449 USA 888.538.7828 (001) 865.938.7828 www.derooyal.com

MedEnvoy Global B.V. Prinses Margrietplantsoen 33 – Suite 123 2595 AM the Hague The Netherlands

PART#445-00013G I REVISED 10/23

©2022 DeRoyal Industries, Inc. All Rights Reserved. DeRoyal and the DeRoyal logo are registered trademarks of DeRoyal Industries, Inc.

ELECTROSURGICAL PENCIL AND ELECTRODES

Table with 2 columns: Symbol and Text. Symbols include MD, STERILE/O, RX ONLY, and a crossed-out symbol. Text includes 'MEDICAL DEVICE', 'STERILIZED WITH ETHYLENE OXIDE', 'NOT MADE WITH NATURAL RUBBER LATEX', and 'FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.'

The rocker and push button tips are intended for use with any active electrode with a 3/32 inch (2.4 mm) diameter shaft...

USE To cut and coagulate by means of high frequency electrical current during electrocauterium procedures with use of electrocauterium units.

INDICATIONS This device may be used during any procedure where electrocauterium is indicated.

- WARNING Read and understand all warnings, cautions and directions completely and carefully before use. Failure to do so may result in serious injury to either the patient or the user.

- Read and understand all warnings, cautions and directions completely and carefully before use. Failure to do so may result in serious injury to either the patient or the user. This device is intended to be used by licensed practitioners familiar with electrocauterium. Use requires a thorough understanding of electrocauterium principles and procedures to avoid shocks and burns to the user, patient, and staff.

- Some anatomical locations may present risk of injury. Areas of concern include sites near the eye and any site or mass with a narrow shaft or base such as the throat, finger, or large papilloma.

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hemostats, other instruments, or the operating table. Do not lean on the patient, table, or other objects during the procedure. This may cause serious injury to the user, staff, or patient.

- flammable materials, such as gauze or surgical drapes, especially those saturated with oxygen; flammable gases or vapors, such as: concentrated oxygen, nitrous oxide, gases naturally occurring or pooling in the body cavity or depressions, such as in the bowel, the vagina, or umbilicus, flammable anesthetics, or alcohol-based skin prepping agents and tinctures; flammable agents for cleaning, disinfecting, or dissolving adhesives; or other flammable substances.

- Medical electrical equipment should be set up following the provided directions, serviced, and maintained prior to any surgical procedure. Inspect set up may result in arcing. Smoke generated during electrocauterium procedures may be harmful to surgical personnel. These studies recommend using a surgical mask and adequate ventilation of the smoke using a smoke evacuator or some form of smoke extraction.

- Verify the compatibility of all instruments and accessories. Generator and crayon sets are compatible with CEI 60601-1 and IEC 60601-2-2 standards are compatible. Do not use power setting, active accessory or other equipment that may have a maximum output voltage of more than 4000 volts peak to peak.

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loop or wire electrode, otherwise the loop or wire may break.

- CAUTION Low power output or failure of the electrocauterium equipment to function properly at the usual settings may indicate a faulty connection of one or more components in the electrocauterium system. Check the active and dispersive electrodes and their cables and connections, and the generator before increasing the power setting.

- Remove debris from the electrode as needed during surgery. Use a moistened sponge or towel to clean coated electrodes. Do not use abrasive materials, scratch pads, or hard scrubs on a coated electrode. This may damage the coating, creating risk of injury to the patient. Abrasive materials or scratch pads can be utilized with all non-coated electrodes. Do not expose device to acids or other corrosive liquids.

- Directions for Use During the Procedure 1. Read the WARNINGS and CAUTIONS thoroughly and complete the steps described prior to the procedure.

- Storage and Transport Conditions 1. KEEP DRY 2. KEEP AWAY FROM SUNLIGHT 3. RELATIVE HUMIDITY: 10% TO 75% 4. TEMPERATURE: -25°C TO +55°C

- WARRANTY DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship.

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