EN: CRYSTALLINE® CRITICAL MEASUREMENT CANISTER

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Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#660931 | REVISED 5/2024

©2024 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Crystalline are trademarks or registered trademarks of DeRoyal Industries, Inc.

CRYSTALLINE® CRITICAL MEASUREMENT CANISTER

	DO NOT REUSE
	MEDICAL DEVICE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of product.

DESCRIPTION

The DeRoyal® Crystalline® Critical Measurement Canister is a waste fluid collection and disposal canister. The canister has a capacity of 300cc and is used when accurate measurements of aspirated fluids are desired. It has volume markings in 5cc increments with an accuracy of +/- 10% accuracy at 0-50cc and 5% accuracy at 55cc-300cc. The canister is hooked up to tubing that is connected to the vacuum system in the wall. Patient tubing is then connected to the "Patient Port" to allow for instruments (i.e. Yankauer) to be attached to evacuate fluids from surgical and/or clinical site(s).

INTENDED USE

The DeRoyal® Crystalline® Critical Measurement Canister is intended to aid in the management and disposal of waste fluid during a variety of medical procedures.

CONTRAINDICATIONS

- Not intended for long-term storage of bodily

- fluids.
- Fluids that may enter the system should not be reintroduced into the body.
- Not recommended for pleural drainage or liposuction. A specially designed canister must be used for pleural drainage to prevent lung collapse or other injury.

WARNINGS

- Product vacuum resistance tested to 27 inHg (685mmHg). **DO NOT EXCEED THIS PRESSURE.**
- Minimum recommended pressure: 30 mmHg.
- Inspect the canister for visible cracks or broken components. The rim should be smooth and free of any burrs to ensure proper function. Replace at healthcare professional's discretion.
- DO NOT** use lid if filter is or has been wet as product will not suction.
- DO NOT** reuse or attempt to sterilize.

CAUTIONS

- Ensure all connections are secure. Improper setup may result in product malfunction or loss of suction.
- Avoid tilting, which can result in premature shutoff or inaccurate volumetric readings. The product should be maintained on a level surface.
- Induce vacuum into canister only at the beginning of procedure. Shut off vacuum when not in use. For optimal performance, it is recommended that the suction tubing used is no less than 6mm (~1/4") internal diameter and no longer than 6 meters (~20 feet) in length. However, tubing with a 4.7mm (3/16") internal diameter may be used.
- All measurements are approximate.
- When handling the Crystalline® canister,

utilize universal personal protective equipment including gloves, face mask, and proper clothing to ensure protection from splashes during transport and disposal of aspirated fluid in the container.

DIRECTIONS

- Assemble lid to canister by snapping lid in place on rim of canister body so lid is firmly seated around the entire circumference of the canister.
- Slide canister into bracket.
- Attach vacuum tubing to "Vacuum Port."
- Attach patient tubing to "Patient Port."
- Attach other end of patient tubing to the patient suction device (e.g., Yankauer).

TO REMOVE CRITICAL CARE MEASUREMENT DEVICE

- Remove patient tubing from "Patient Port" of the critical measurement canister.
- Remove the vacuum tubing from "Vacuum Port" of the critical measurement canister and cap off all ports.
- Gently lift the critical measurement device upward out of its bracket.

TO DISCARD CRITICAL CARE MEASUREMENT DEVICE

- Discard the critical measurement device intact, ensuring all ports are capped-off.
- OR**
- Twist and remove pour spout cap.
 - Dispose of drainage and disposable liner in compliance with facility's guidelines.

STORAGE AND TRANSPORT CONDITIONS

DO NOT store product at extreme temperatures or in a moist damp environment.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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