EN:HYDRO-TEMP® NEONATAL SKIN TEMPERATURE PROBE COVER

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EN: ENGLISH HYDRO-TEMP® NEONATAL SKIN TEMPERATURE PROBE COVER

	SINGLE PATIENT USE
	NON-STERILE
	MEDICAL DEVICE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of product.

INTENDED USE

The DeRoyal® Hydro-Temp® Neonatal Skin Temperature Probe Covers are used to adhere the thermistor of the temperature probe to the patient and insulate the thermistor from ambient room temperature ensuring correct temperature readings.

! WARNINGS

- This product is for use only in health care facilities and only by persons with specific training and experience in use of the device.
- Due to the possibility of false readings and possible patient injury, **DO NOT** remove probe or modify the probe in any way.
- If the sensor-to-skin contact is lost, it should be reestablished immediately to avoid conditions that may cause full or incorrect heat output.
- NEVER** shut off the warmer's or incubator's safety alarms.
- These products are intended to be used with DeRoyal-branded NICU neonatal skin temperature probes. They have not been evaluated for use with other manufacturer's products.
- While adverse reactions are rare, the following have been associated with use of skin temperature sensors during use or immediately after use: electrical burns and skin irritation or inflammation.

CAUTIONS

- Discontinue use if an allergic reaction is observed from use of the adhesive pad cover.

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- Consider a less aggressive adhesive pad if skin irritation results upon removal of the cover.
- The adhesive cover should never be applied to a skin surface which has been damaged by abrasion, laceration, incision, or burning.

DIRECTIONS

- Carefully remove the temperature probe and probe cover from the skin. Drops of sterile water may be used on the adhesive or gel if needed.
- Discard old temperature probe cover.
- Remove new probe cover from package and peel off backing.
- Center temperature probe (sensor) on new cover.
- Prep skin for new probe cover. Remove body oils and hair.
- Carefully place probe in accordance with infant warmer and/or incubator manufacturer's recommended site location.
- Take care to ensure good sensor-to-skin contact.
- After reapplying probe cover to skin several times, it may be necessary to replace with a new probe cover.
- When temperature monitoring is completed, gently remove the adhesive cover from the patient's skin. Drops of sterile water may be used as needed to assist in releasing the adhesive.

STORAGE AND TRANSPORT CONDITIONS

Do not store product at extreme temperatures or in a moist/damp environment.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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