



## **Polyderm™** **Polyderm™ Border** **Polyderm™ Plus**

**Do not reuse • Disposable • Sterile**

### **IMPORTANT**

Read carefully and completely before using or applying.

### **GENERAL**

- Polyderm™ is a sterile polyurethane, non-adherent hydrophilic foam dressing that is available without adhesive border (Polyderm™), with stretchable fabric tape border (Polyderm™ Border), and with border and breathable barrier film for protection from external contaminants (Polyderm™ Plus).
- Polyderm™ products are effective in the management of exudating wounds. The thick, soft construction is conformable and comfortable making it ideal for painful wounds requiring protection and cushioning.

### **INTENDED USE**

Polyderm™ products are indicated for dermal ulcers (pressure ulcers, leg ulcers, secreting lesions), second degree burns, donor sites, lacerations, cuts, abrasions, tube sites and other superficial wounds.

### **DIRECTIONS FOR USE OF POLYDERM™**

1. Cleanse the wound of excess exudate and thoroughly dry skin around wound.
2. Center the polyurethane foam over wound.
3. Secure the nonbordered foam dressing in place

with appropriate retention device such as Apex conforming gauze, gauze roll, or tape.

4. Leave the dressing in place (not more than two-three days) unless it is uncomfortable, leaking or there are clinical signs of infection.
5. To remove Polyderm™, remove the retention dressing or tape to gently lift the polyurethane foam from the wound site.
6. Dispose of soiled dressing per institutional guidelines.

### **DIRECTIONS FOR USE OF POLYDERM™ BORDER AND POLYDERM™ PLUS**

1. Cleanse the wound of excess exudate and thoroughly dry skin around wound.
2. Peel back paper to release liner from the back of the Polyderm™ Border or Polyderm™ Plus dressing.
3. Center the polyurethane foam over wound and press foam and tape in place.
4. Leave the dressing in place (not more than two-three days) unless it is uncomfortable, leaking or there are clinical signs of infection.
5. To remove Polyderm™ Border or Polyderm™ Plus, gently and carefully lift the edges of the tape border and peel the dressing off.
6. Dispose of soiled dressing per institutional guidelines.



**Contents sterile unless individual package has been opened or damaged.**

**Store at room temperature and avoid excessive heat and humidity.**



**Manufactured by DeRoyal**  
200 DeBusk Lane  
Powell, TN 37849 U.S.A.  
888.938.7828  
865.938.7828

Part #714530  
©2005 DeRoyal. All Rights Reserved.  
Revised 11/05