SKIN TEMPERATURE SENSOR

SENSOR DE TEMPERATURA CUTÁNEA

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SENSOR DE TEMPERATURA DA PELE

Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#74-14902

Revised 2/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo and Improving Care. Improving Business. is a registered trademark of DeRoyal Industries, Inc.

Mindray is a registered trademark of Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Draeger is a registered trademark of Draegerwerk AG & CO. KGaA Corporation.

SKIN TEMPERATURE SENSOR

	DO NOT RESTERILIZE
	DO NOT REUSE
	STERILIZED WITH ETHYLENE OXIDE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury.

INTENDED USE

The DeRoyal® Skin Temperature Sensor is to be used for routine monitoring of a patient's skin surface temperature.

PRODUCT DESCRIPTION

The Adult Skin Sensor consists of a temperature-sensing wire set with an adhesive probe cover that facilitates application to the patient's skin surface. The temperature monitoring sensor located at the distal end of the wire is affixed to the center of the disk-shaped foam pad, which has a reflective backing and an adhesive layer to hold the sensor in place.

The temperature monitoring sensor is located at the tip of the probe. The blue connector at the proximal end interfaces with DeRoyal-branded cables for connection to a monitoring unit to display temperature. The temperature monitoring sensor is available in YSI 400 or YSI 700 series configurations for connection to a compatible monitoring unit. Whether the probe is a 400 or 700 series configuration is indicated on the device labeling. Consult the user manual for the patient monitor to ensure compatibility. The probes are designed to interface with DeRoyal-branded cables for connection with YSI 400 or 700 series compatible monitors, including the following patient monitors and equivalent models: Mindray® Passport, Philips IntelliVue®, Siemens/Draeger Infinity®, and GE Datex-Ohmeda® brands.

ADVERSE REACTIONS

While adverse reactions are rare, the following have

been associated with use of the skin temperature sensor during use or immediately after use: electrical burns and skin irritation or inflammation.

! WARNINGS

- The Skin Temperature Sensor should only be used on healthy adult skin.
- DO NOT** apply the adhesive cover on this probe to a skin surface which has been damaged by abrasion, laceration, incision, burning, irritation, chafing or inflammation.
- DO NOT** use this skin probe on neonatal patients.
- DO NOT** use probe without probe cover. Use without probe cover can cause inaccurate readings, and could potentially lead to patient injury.

CAUTIONS

- Device is intended for use by a licensed practitioner.
- Take care to ensure the cable and connector do not get wet and proper techniques are used during electrosurgical procedures to reduce radio frequency interference current and potential burns to the patient.
- An adequate electrosurgical dispersive ground electrode close to the active surgical site should be properly connected.
- DO NOT** intertwine the cables, especially the monitor cables, with the electrosurgical unit's cables.
- The operation of the patient temperature monitor may be temporarily affected during electrosurgical activations. Unusual temperature readings should be checked.
- If temperature does not read properly, check all connections. If problem persists, discard and use a new sensor.
- Check the product for any damage, contaminants, or rough edges before use.
- DO NOT** use an expired device.
- In case of adverse skin reaction, discontinue use.
- DO NOT** use probe without probe cover. Use without probe cover can cause inaccurate readings, and could potentially lead to patient injury.
- The skin probe is intended to be used with DeRoyal-branded interface cables and probe cover. Use with an incompatible cable may affect performance.

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MR COMPATIBILIDAD

- The DeRoyal® Skin Probe has not been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of the Skin Probe in the MR environment is unknown. Scanning a patient who has this medical device may result in patient injury.

DIRECCIONES FOR USE

- Prepare the skin for adhesive pad by removing body oils and hair.
- Verify the compatibility of the probe, interface cable, and patient monitor before use.
- Remove the Skin Temperature Sensor from its package.
- Peel off release liner and apply sensor with probe cover to patient's skin.
- Connect the temperature probe to the appropriate reusable cable from DeRoyal. Connect the cable to the temperature monitor. Secure the cable using the drape clip.
- Follow the directions for use of the patient temperature monitor instrument.
- When temperature monitoring is completed, carefully remove adhesive cover from patient's skin.
- Disconnect the probe (at the connector) and discard the probe. To disconnect, grasp both connectors firmly and pull. **DO NOT** pull on cable or wire.
- Dispose of probe according to facility protocol.

OPERATING SPECIFICATIONS

- Rated output range: 25° to 45° Celsius
- The reference body site is the skin surface temperature.
- The measuring site(s) are intact skin.
- Accurate to +/- 0.2° Celsius
- Time response:
 - The temperature probe has a direct mode of operation.
 - The maximum heating transient time is 12 seconds
 - The maximum cooling transient time is 10 seconds

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STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT
	DO NOT USE IF PACKAGE IS DAMAGED
	RELATIVE HUMIDITY: 85% NON-CONDENSING
	STORAGE TEMPERATURE: - 25° TO + 55° C
	OPERATING TEMPERATURE: 25° TO 45° C

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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