TEMPERATURE PROBE INTERFACE CABLES

	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Interface Cable is a reusable interface cable for use only with disposable temperature probes.

WARNINGS

- DO NOT use if packaging is damaged or compromised.

DIRECTIONS FOR USE

- Connect the Interface Cable into the appropriate monitor jack.
- Connect a disposable temperature probe to the cable.
- Refer to the instruction sheet insert found inside the disposable temperature probe boxes for **SUGGESTED DIRECTIONS FOR USE**.
- After use with probe, disconnect from probe by pulling the two connectors apart. **DO NOT** pull on the cable lead wire while attempting to disconnect. Avoid crushing the cable lead wire or connector.
- Follow facility approved protocol for cleaning of the cable. If cable is damaged, dispose of according to facility protocol.

CLEANING AND/OR MAINTENANCE

- DO NOT autoclave the cable or gas sterilize if temperature will exceed 60 degrees C (140 degrees F).
- DO NOT immerse the cable in liquids for cleaning. If

necessary, clean the cable by wiping it using a cloth moistened with mild detergent.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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