SENSOSTRIP™ TEMPERATURE TREND INDICATOR

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Distributed By:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#74-31188
Revised 8/2022
©2022 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo,
Improving Care. Improving Business. and Sensostrip
are trademarks or registered trademarks of DeRoyal
Industries, Inc.

SENSOSTRIP™ TEMPERATURE TREND INDICATOR

	DO NOT REUSE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of product.

INTENDED USE

The DeRoyal® Sensostrip™ Indicator is intended to be used by a licensed practitioner to measure human forehead surface temperature and monitor temperature trends.

WARNINGS

- For external use only
- Sensostrip™ Indicator is not as accurate for determining the actual core body temperature as a clinical (oral/rectal) thermometer. Instead it is designed to be an unbreakable, quick reading device for temperature screening between uses of a clinical thermometer.

CAUTIONS

- Environmental temperatures might affect the performance of this product

DIRECTIONS FOR USE

- Sensostrip™ Indicator is to be used on the forehead area to estimate the core body temperature by measuring the temperature of the forehead.
- Accurate to $\pm 1.0^{\circ}\text{F}/\pm 0.6^{\circ}\text{C}$, the readings on the Sensostrip™ Indicator have been adjusted to approximate oral temperature readings during normal to febrish conditions.
- Remove the backing liner and apply to the center of the forehead, just above the eyebrows.
- Read the indicator when color stops changing, and while it is still on the forehead.
- Green indicates the correct temperature; if green does not appear, the temperature is between blue and tan.
- If you think your temperature is different from that shown by the Sensostrip™ Indicator, you should check by using a clinical thermometer. If fever is present, consult your physician for appropriate treatment.
- Discard after use.

STORAGE AND TRANSPORT CONDITIONS

Do not store product at extreme temperatures or in a moist/damp environment.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

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WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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