DERMANET AG® WOUND DRESSING
DERMANET AG® BORDER DRESSING

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Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#757345
Revised 6/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care. Improving Business., Multipad, Sofsorb, Polyderm, Stretch Net, Algidex Ag, Dermanet, and Dermanet Ag are trademarks or registered trademarks of DeRoyal Industries, Inc.

INSTRUCTIONS FOR USE

APPLICATION OF DERMANET AG®+ WOUND DRESSING

1. Thoroughly cleanse wound.
2. Remove Dermanet Ag®+ dressing from the backing material.
3. Place Dermanet Ag®+ dressing over shallow wounds or pack into deep wounds.
4. Cover with appropriate secondary dressing (e.g. gauze, Multipad™ dressing, Sofsorb® dressing, Polyderm™ dressing, etc...) based upon amount of wound drainage.
5. Secure dressing in place as needed with tape, roll gauze or Stretch Net™ dressing.
6. Dermanet Ag®+ dressing can be worn up to five days depending on the amount of exudate and condition of the wound or until the secondary dressing is saturated and requires changing.

REMOVAL OF DERMANET AG®+ WOUND DRESSING

1. Remove secondary cover dressing.
2. Gently remove Dermanet Ag®+ dressing from wound. Irrigate wound with saline as needed to help loosen dressing.
3. Continue to thoroughly cleanse wound to remove wound drainage or any residue left by the Dermanet Ag®+ dressing.

APPLICATION OF DERMANET AG®+ BORDER DRESSING

1. Clean and thoroughly dry site.
2. Simultaneously peel release liner from center of dressing while aligning the Dermanet Ag+ dressing to the center over the site.
3. Press dressing into place.
4. Remove the "Top Clear Carrier Film" by lifting away at the tab.

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REMOVAL OF DERMANET AG®+ BORDER DRESSING

1. Gently grasp a corner on the edge of the dressing and slowly pull the dressing from the skin in the direction of hair growth.

STORAGE AND TRANSPORT CONDITIONS

Store this product in a cool, dry place. Dispose of this product in accordance with local, state, and federal guidelines.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT
	DO NOT USE IF PACKAGE IS DAMAGED

Contents sterile unless package has been opened or damaged.

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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DERMANET AG® WOUND DRESSING
DERMANET AG® BORDER DRESSING

	USE BY DATE
	DO NOT REUSE
	STERILIZED USING IRRADIATION
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

GENERAL

Dermanet Ag®+ wound dressing is a combination of Dermanet® wound contact layer and Algidex Ag® Silver technology. The Dermanet® wound contact layer is an inert, non-adherent material that helps to shield and protect fragile granulation tissue. It helps to reduce trauma and pain that can be caused during dressing changes. The Algidex Ag® technology is a unique formulation of ionic silver combined in an alginate and Maltodextrin matrix. The Algidex Ag® formulation provides immediate and extended release of active ionic silver for broad antimicrobial effectiveness and helps to prevent contamination from external bacteria. The Algidex Ag® formulation decreases surface wound contaminates, decreases wound odor and creates a moist environment conducive to healing. The Maltodextrin creates an environment that helps the body's own cells to carry out the task of granulation tissue formation while eliminating wound odor.

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Local wound care cannot overcome the deficits of unrelieved malnutrition, pressure or trauma, or compromised blood flow. Interventions must be instituted to reduce/relieve these factors that contribute to impaired wound healing in order to achieve the optimum outcome.

INTENDED USE

Dermanet Ag®+ dressings can be used on infected and non-infected wounds, including dermal ulcers (e.g. leg ulcers, pressure ulcers), diabetic ulcers, abdominal wounds, superficial wounds, lacerations, cuts, surgical incisions, abrasions, donor sites, and burn wounds.

INDICATIONS

Dermanet Ag®+ dressings are used for the management of multiple wounds including:

- Pressure Ulcers
- Venous Ulcers
- Dermal lesions (or secreting skin injuries)
- Burn injuries
- Donor sites
- Lacerations
- Surgical Incisions
- Vascular Access Sites

CONTRAINDICATIONS

Dermanet Ag®+ dressings should not be used on:

- Patients with known sensitivity to alginates
- Third degree burns

CAUTIONS

- When visible signs of infection are present in the wound, it is recommended to address the cause of the infection with proper systemic medical treatment as needed along with topical application of the Dermanet Ag®+ dressing.
- Dermanet Ag®+ dressings are for external use and may not be surgically implanted.
- Discontinue use if signs of sensitivity appear.

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APPLICATION DU PANSEMENT AVEC BORDURE DERMANET AG®+
REMOÇÃO DO CURATIVO DE BORDA DERMANET AG®+
RETRAIT DU PANSEMENT AVEC BORDURE DERMANET AG®+