Application Video

How the BOA®
 Fit System works.




For questions or concerns regarding your DeRoyal spine brace, please contact our customer service line at

1(800)-DEROYAL or customerservice@deroyal.com.

They will direct you to a dedicated spine specialist that can address your needs or answer questions regarding your spine brace.

PUTTING ON YOUR BRACE:

1. Ensure both dials are released and in the “pulled out” position 
2. Wrap the brace belts around your torso and align the back panel to the center of your back (stand with your back against a wall to prop up the brace to assist, if needed).
 The gray face of the left belt should attach to the front panel gray hook (if not already attached). See illustration A & B.
3. Wrap the right side of the belt over the left side using the hand strap and ensure the right oval belt front completely overlaps the left oval belt front.
4. Raise or lower the sternal pad bar as needed by pinching the adjustment mechanism (A) and sliding the bar up or down. The top of the chest frame should be positioned ~4” below your collarbones and in the center of your chest.
5. Buckle the shoulder straps to the sternal pad and tighten, if needed.
6. Push dials in and turn clockwise evenly to provide compression.  
 The brace should fit snug but not too tight.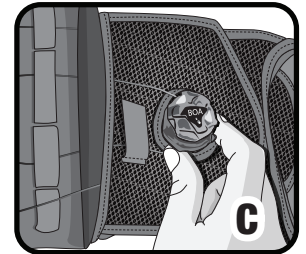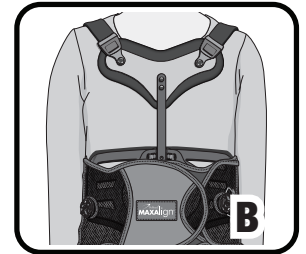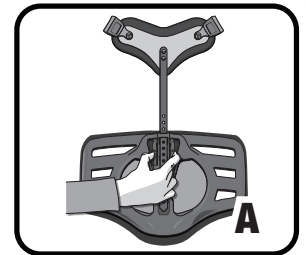

TAKING OFF YOUR BRACE:

1. Pull out on both dials to release compression 
2. Unbuckle the right shoulder buckle and use the hand strap to open the brace. Slide the brace off and set aside until it is needed for future use.

**PUSH IN
 TO ENGAGE**

**TIGHTEN
 TURN FOR
 PRECISION FIT**

**PULL UP FOR FAST
 RELEASE**

Care Instructions:

Remove panels from the brace and clean all soft goods by hand washing with mild soap and let air dry completely prior to reapplication. Anterior and posterior panels can be cleaned by wiping down with a disinfectant wipe.

This Quick Reference Guide does not supersede the instructions given by a prescriber or the information contained in the Instructions for Use (IFU) which accompany this device. Please refer to the IFU for all warnings, cautions and complete instructions.

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

©2023 DeRoyal Industries, Inc. | All Rights Reserved.
 The DeRoyal logo and MAXAlign are registered trademarks of DeRoyal Industries, Inc.
 Boa is a registered trademark of Boa Technology Inc.
 Part # 0-2201 | Reprint # 0-2572 | Revised 5/2023

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